wi-Tens

*Wireless
remote control*

Manuale d'uso

User manual

Manual de uso

New Age
Medical Devices

DECLARATION OF CONFORMITY

The undersigned New Age Italia s.r.l.
Via De Brozzi, 3 – 48022 Lugo (RA)
Tel: +39 0545 32019 – Fax +39-0545 369028

in the person of its legal representative, Mr. Barnabè Manuel declares that
the following **class IIa** device:

Brand Name: New Age Italia
Model: Wi-TENS

is up-to the requisites prescribed in EEC Directive 93/42CEE modified by Directive 2007/47CE concerning health regulations and absorbed by Legislative Decree no. 37 dated 25/01/2010;

that the procedure for the evaluation of conformity described in Annex II to the above-mentioned Directive has been duly performed;

The notified authority N°0476 has, in virtue of the certificate HD 600390570001, issued a permission to apply a CE conformity mark to the above-mentioned regulations prescribed in EEC Directive 93/42CEE and modified by Directive 2007/47CE absorbed by Legislative Decree no. 37 dated 25/01/2010.

Lugo, 03/11/2013

The company's legal representative

An official circular stamp of New Age Italia SRL. The text inside the stamp includes: "NEW AGE ITALIA SRL", "Sede legale: Via Gramsci n. 206", "48013 Castel Maggiore (BO)", "Sede operativa: Via De Brozzi n. 3", "48022 Lugo (RA)", "C.F. 02299101200", and "E-mail: info@newageitalia.it". There is a signature over the stamp.

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CONTENTS

INTRODUCTION	24
IMPORTANT SAFETY PRECAUTIONS AND WARNINGS	26
FEATURES	29
PRODUCT STRUCTURE	30
REMOTE CONTROL	32
BATTERY INFORMATION	32
TREATMENT INFORMATION	34
CLEANING AND STORAGE INFORMATION	36
TECHNICAL SPECIFICATION	37
PROGRAM	38
TROUBLESHOOTING	39
WARRANTY	40
DISPOSAL	59
EXPLANATION OF SYMBOLS	59

INTRODUCTION

Thank you for purchasing New Age Italia Wireless TENS wi-Tens for pain relief solution. Please read the complete manual carefully before using the device for the first time and keep this instruction manual in a convenient place or store it with the device for future reference.

wi-TENS is a TENS stimulator. What is TENS?

Transcutaneous Electrical Nerve Stimulation (TENS) is a noninvasive, drug free method of controlling pain. TENS uses tiny electrical impulses sent through the skin to nerves to modify your pain perception. TENS does not cure any physiological problem; it only helps control the pain. TENS does not work for everyone; however, in most patients it is effective in reducing or eliminating the pain, allowing for a return to normal activity.

How does TENS work?

Scientific theory suggests that electrical stimulation therapy may work in several ways:

- the gentle electrical pulses move through the skin to nearby nerves to block or shut out the pain message from ever reaching the brain from the source of the pain;
- the gentle electrical pulses increase the production of the body's natural pain killer, such as endorphins;
- furthermore, it is thought that the electrical stimulation improves blood circulation as well. Muscles contract and relax with the flow of the electrical stimulation. With repeated contracting and relaxing action, the blood flows in and out and the blood circulation is improved.

Intended uses include the following:

- temporary relief of pain associated with sore;

INTRODUCTION

- aching muscles in the shoulder, waist, back, neck, upper extremities (arm);
- lower extremities (leg) due to strain from exercise;
- lower extremities (leg) due to normal household work activities.

The Wireless TENS stimulator should be applied to normal, healthy, dry and clean skin of adult patients.

The wi-Tens contains the following components:

- 6 x AAA batteries;
- 2 x Wireless TENS Stimulator;
- 2 x electrodes wi-Tens;
- 1 x Remote Control;
- 1 x User manual.

IMPORTANT SAFETY PRECAUTIONS AND WARNINGS

It is important that you read all the warnings and precautions included in this manual because they are intended to keep you safe, prevent injury and avoid a situation that could result in damage to the device.

SAFETY SYMBOLS USED IN THIS MANUAL

 WARNING	Indicates a potentially hazardous situation which, if not avoided, could result in serious injury.
 CAUTION	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or cause damage to the device or other property.

WARNING

To avoid damage to your health, please make sure that you do not use the device in the following situations:

- pacemakers;
- electronic life support equipment, such as respirators;
- electronic medical devices attached to the body, such as electrocardiographs;
- on open wounds or rashes, or over swollen, red, infected, or inflamed areas or skin eruptions (e.g.,

IMPORTANT SAFETY PRECAUTIONS AND WARNINGS

phlebitis, thrombophlebitis, varicose veins); On top of, or in proximity of, cancerous lesions.

- over areas of skin that lack normal sensation;
- pregnant women;
- children or infants;
- persons incapable of expressing their thoughts or intentions;
- while having a bath or shower;
- while sleeping;
- while driving, operating machinery, or during any activity in which the electrical stimulation can put you at risk of injury;
- the stimulation should not take place while the user is connected to high-frequency surgical equipment, it may cause burn injuries on the skin under the electrodes, as well as problems with the stimulator;
- do not use the stimulator in the vicinity of shortwave or microwave therapy equipment, since this may affect the output power of the stimulator.

PAIN MANAGEMENT WARNINGS

- if you have had medical or physical treatment for your pain, consult with your physician before using this device;
- if your pain does not improve, becomes seriously chronic or severe, or continues for more than five days, stop using the device and consult with your physician;
- The mere existence of pain functions as a very important warning telling us that something is wrong.

IMPORTANT SAFETY PRECAUTIONS AND WARNINGS

Therefore, if you suffer from any serious illness, consult your physician in order to confirm that it is advisable for you to use this TENS Stimulator.

NEVER APPLY THE PATCH TO:

- The head or any area of the face. (e.g. eyes or mouth);
- the front part of the neck (especially the carotid sinus) or any area of the throat, because this could cause severe muscle spasms resulting in closure of the throat, difficulty in breathing, or adverse effects on heart rhythm or blood pressure;
- both sides of the thorax or upper back simultaneously (lateral or front and back), or across your chest or your heart because the introduction of electrical current may cause rhythm disturbances. The heart area must be kept outside the action of the electrostimulation area.

PRECAUTIONS REGARDING THE DEVICE

- Do not bend or fold, and place it on the plastic film and then store into the sealed package when not in use;
- do not apply ointment or any solvent to the device or to your skin;
- the device is already pre-gelled and will adhere to your skin;
- to avoid damage to the adhesive surface of the pads, put the pads only on the skin or on the plastic film provided;
- make sure it is connected well with your skin.

IMPORTANT SAFETY PRECAUTIONS AND WARNINGS


DO NOT USE YOUR PATCH THIS WAY

- Patches of the device should not touch each other when placed onto your skin;
- do not place it on your spine or backbone;
- do not place or relocate the device while it is working;
- take off the device from your skin after treatment;
- do not share the patch with others. It is a single person use;
- always turn the power off before removing or changing the pad location;
- if the stimulator is not functioning properly or you feel discomfort, immediately stop using the device;
- do not use for any other purpose except for what it is intended for;
- do not throw the batteries into a fire. The batteries may explode;
- use the accessories recommended by manufacturer.


FEATURES

- Mini appearance;
- wireless transmission;
- fast and convenient adjustment;
- single patient use, clean and hygienic;
- one therapy mode consists of three different therapy phases.

PRODUCT STRUCTURE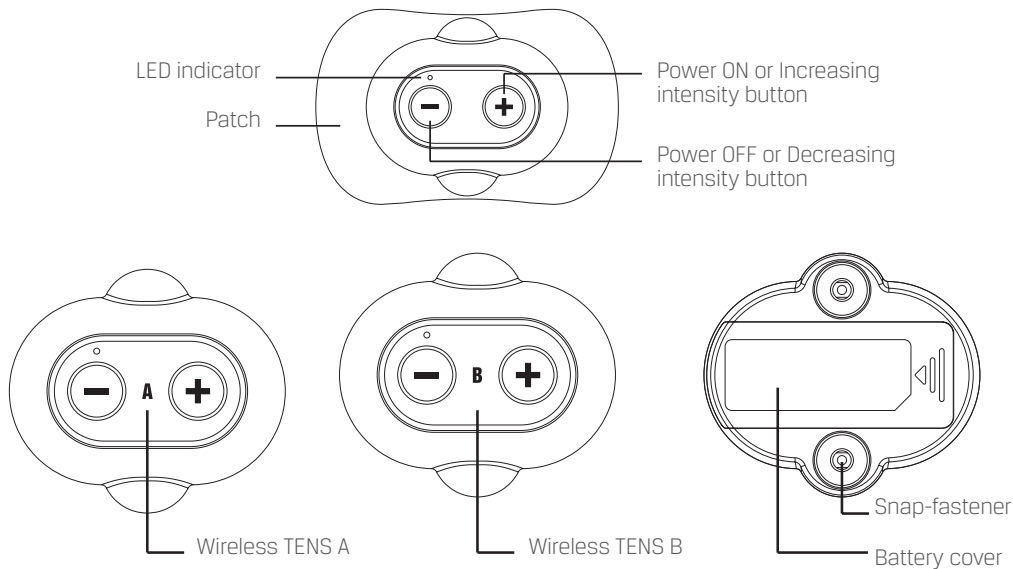

PRODUCT STRUCTURE

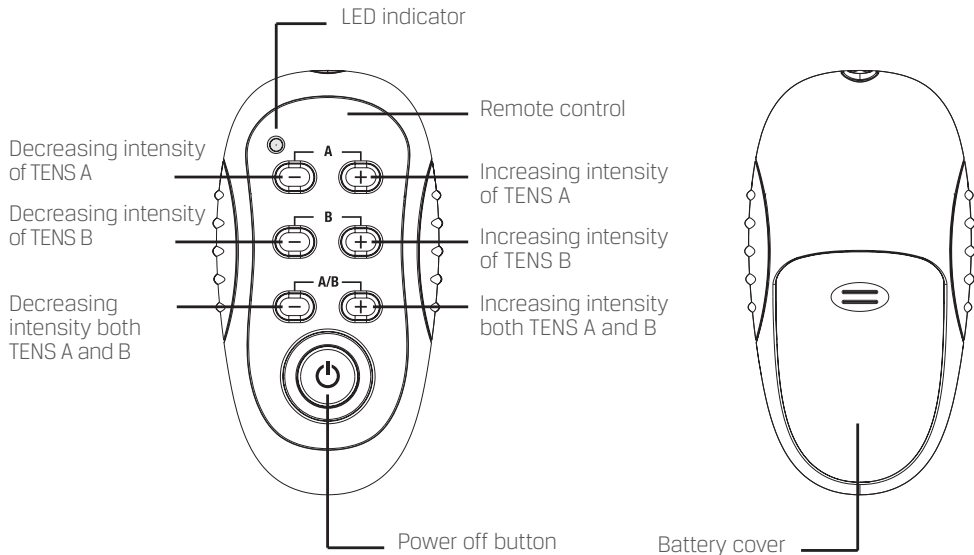REMOTE CONTROL



As a main feature of wi-Tens, the remote control allows you to adjust output intensity of both wireless TENS A and B through wireless transmission. Before using the remote control, please make sure that wireless TENS is supplied with two 1.5V AAA batteries.

CAUTION


1. Keep the remote control far away from children;
2. For details of battery replacement, please refer to the battery information on next page.

BATTERY INFORMATION


1. Before replacing, please ensure the device has been turned off.
2. Take off the device from the patch.
3. Push out the battery cover.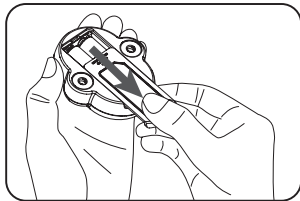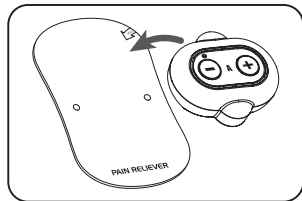

4. Insert the batteries (2 x 1.5V AAA).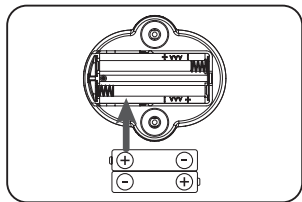

5. Close the battery cover.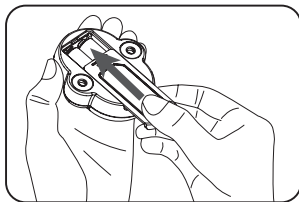


WARNING

- Remove the batteries if the device is not used for a long time;
- keep batteries out of the reach of children;
- dispose the used batteries safely according to the local regulations;
- do not mix old and new batteries or different types of batteries;
- in case batteries leak and they come into contact with the skin or eyes, wash immediately with copious amounts of water.

Standard for battery replacement


If you feel the stimulation intensity is weak or the device cannot turn on, the batteries may be exhausted, please replace the batteries.

TREATMENT INFORMATION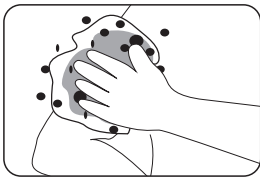

STEP 1 - CLEANING OF SKIN



Clip excess hair from painful area. Wash area with soap and water and dry completely.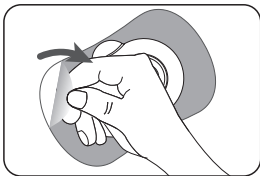

STEP 2

PREPARATION OF THE WIRELESS TENS

Connect the device to the patch. Remove the electrodes from the clear plastic film. Save the plastic film for storing the electrodes after use.

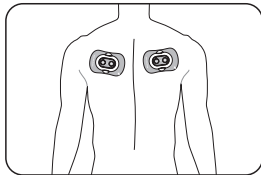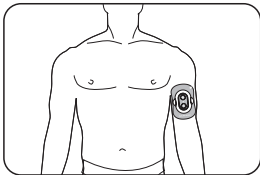STEP 3 - PLACING THE WIRELESS TENS

Place the Wireless TENS on a clean, dry and healthy skin above and below or on either side of the painful area.

TREATMENT INFORMATION

NOTE:

1. Make sure that it is in contact well with your skin.
2. You can choose to use one or two of the Wireless TENS.

STEP 4 - TURN ON THE DEVICE

Press the "  " button to turn on the device.

NOTE:

The LED indicator on the device has four conditions:




1. Light, the device is ready for use;
2. Blinking, the device is working;
3. Rapid blinking, receiving information from the remote control;
4. Light off, the device is turned off.

STEP 5 - SET OUTPUT INTENSITY

Using either the remote control or buttons on the device, set the intensity level by using the "  /  " buttons.

NOTE:

During treatment, if the LED indicator stops blinking, please check whether the device is in perfect contact with your skin.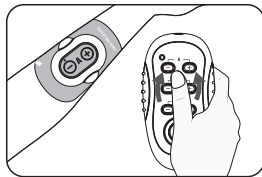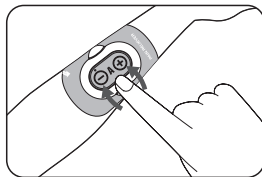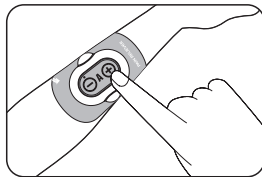

TREATMENT INFORMATION

STEP 6 - TURN OFF THE DEVICE

The following ways to turn off are allowed:

1. The device will automatically turn off after 30 minutes treatment.
2. Keep pressing the "⊖" button on the device until the LED indicator is off.
3. The device can be turn off at any time by pressing the "⏻" button on the remote control.



CLEANING AND STORAGE INFORMATION

Cleaning the device:

- 1) Take off the device from the patches.
- 2) Clean device and patch with a lightly moistened cloth and wipe gently.
 - Do not use chemicals.
 - Do not let water get into the internal area.

STORING THE DEVICE:

- 1) Turn off the device and place the patch onto the plastic film.
- 2) Store the box in a cool, dry place, -10°C / 50°C; 10% / 80% relative humidity.
- 3) Keep the device out of the reach of children.
- 4) Remove the battery if the device is not in use for long periods of time.

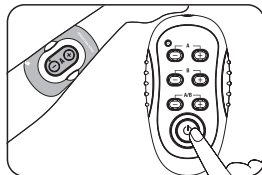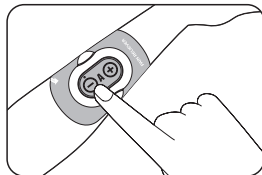CAUTION

- If the patches no longer stick to your skin or the patches are broken, you should replace them;
- the Wireless TENS is single patient use. Don't share it with others.

TECHNICAL SPECIFICATION

TYPE	Wireless TENS (wi-Tens)
POWER SUPPLY	DC3.0V (2×AAA)
WAVE FORM	Bi-phase square pulsed wave
FREQUENCY	5/125Hz
PULSE WIDTH	130/250µs
OUTPUT VOLTAGE	0/35V (at 500 ohm load)
OUTPUT INTENSITY LEVEL	0/20 levels
TREATMENT TIME	30 minutes
OPERATING CONDITIONS	10°C / 40°C; 30%RH / 80%RH
STORAGE CONDITIONS	-10°C / 50°C; 10%RH / 80%RH
SIZE	103 (L) x 55.8 (W) x25.2 (H) mm (Remote control) 60 (L) x 53 (W) x17.7 (H) mm (Wireless TENS A and B)
WEIGHT	42.5g without battery (Remote control) 44.4g without battery (Wireless TENS A and B)

PROGRAM

During use, the Wireless TENS has three phases as following:

PROGRAM	FREQUENCY	PULSE WIDTH	TIME	DESCRIPTION
1	80/125Hz	130 μ s	5min	The frequency is changed in a cycle, which is from 80Hz to 125Hz, then back 80Hz. The cycle time is 32s.
2	5-10Hz	250 μ s	20min	The frequency is changed in a cycle, which is from 5Hz to 10Hz, then back 5Hz. The cycle time is 10s.
3	80/125Hz	130 μ s	5min	The frequency is changed in a cycle, which is from 80Hz to 125Hz, then back 80Hz. The cycle time is 32s.

TROUBLESHOOTING

If the unit does not operate after taking the above-mentioned measures, contact the nearest dealer.

PROBLEM	POSSIBLE CAUSES	POSSIBLE SOLUTION
The device cannot turn on	Battery exhausted	Replace the battery
	Battery not installed correctly	Insert the battery observing polarity
The stimulation is weak or you cannot feel any stimulation	Patches dried out or dirty	Replace with new patches
	Patches cannot stick skin well	Reconnect the patches
The stimulation is uncomfortable	Intensity is too high	Decrease intensity
	Maybe you don't operate the device according to the manual	Please check the manual before use
The LED indicator on the remote control is off	Battery on the remote control exhausted	Replace the battery
The skin becomes red and/or you feel a stabbing pain	Use the Patches on the same site every time	Re-position the Patches
	The patches aren't stuck onto the skin properly	Ensure the patches are stuck securely on the skin
	The patches are dirty	Clean
	The surface of the patches was scratched.	Replace them

WARRANTY

Please contact your dealer or the device centre in case of a claim under warranty. If you have to send in the unit, enclose a copy of your receipt and state what the defect is. The following warranty terms apply:

1. The warranty period for device is two years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Repairs under warranty do not extend the warranty period either for the device or for the replacement parts.
3. The following is excluded under the warranty:
 - All damage which has arisen due to improper treatment, e.g. nonobservance of the user instruction;
 - all damage which is due to repairs or tampering by the customer or unauthorized third parties;
 - damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre;
 - The battery and patches are subject to normal wear and tear.

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Used fully discharged batteries must be disposed of in a specially labelled collection container, in accordance with legal obligations.

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Disposal in accordance with Directive 2002/96/EC and 2003/108EC

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Ente certificatore 0476

Complies with the European Medical Device Directive (93/42/EEC) and amended by directive 2007/47/EC requirements. Certification Body 0476

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NEW AGE HAS BEEN MARKED ACCORDING TO THE DIRECTIVE UNI EN ISO 9001-2008 AND UNI CEI EN 13485-2004. ALL PRODUCTS ARE MADE ACCORDING TO THE CEE 93/42 DIRECTIVE ON MEDICAL DEVICES